GB Sealing after bleaching

Instructions for use

Smoking and drinking coffee or tea are habits that can quickly compromise the result of a teeth bleaching procedure. It is usually recommended to stop smoking and drinking coffee and tea for at least 72 hours after bleaching, when teeth are more likely to absorb new discolorations. Nevertheless it is a very difficult for patients to comply with such a limit.

BlancOne® SEAL is a silicone based, acid protecting varnish for sealing teeth after bleaching procedures. It forms a film that protects teeth from new discolorations from acidic food, discoloring beverages and smoke, thus limiting new discolorations and making teeth bleaching more durable.

BlancOne® SEAL varnish contents Nano-Fluorapatite and Nano-Calciumfluoride. Nano-technology fluorapatite particles are known to enhance remineralization of tooth material.

BlancOne® SEAL is one component and does not require etching. It builds a protective film, that can be renewed or repaired just as you like, if necessary. The application is easy and time saving. One kit is usually enough for approximately 20 patients.

Indication

Sealing teeth after bleaching procedures

Application of BlancOne® SEAL after teeth bleaching

1. Before application vigorous shake the bottle of BlancOne® SEAL.

During shaking listen to the glass bead inside the bottle.

2. Leave lip retractor in place to make application easier.
3. Remove gum protection, in case it was used during bleaching treatment.
4. Use disposable cups and applicators provided in the package, one dip is enough to cover approximately 3 teeth.
5. Close the bottle immediately after use.
6. Apply BlancOne® SEAL directly on clean and dry tooth surface. Avoid contact with the mucosa.
7. Apply BlancOne® SEAL 2-3 times on each tooth in an even thin layer.

If teeth have been bleached on the back, apply the varnish there as well.

BlancOne® SEAL may advantageously be applied to tooth areas exposed by receding gums and in be-

tween teeth.

8. Let it dry for 1 minute or dry it gentle in an oil free air stream.
9. Thoroughly rinse the patient's teeth after completing sealing.

Important remarks

- In the first 2 hours after application of BlancOne® SEAL don't eat anything nor brush the teeth.
- Do not seal teeth if bleaching treatment is not completed! In case of follow-up with take home bleaching treatments, perform sealing after completion of home treatment.
- Air abrasion treatment is recommended before bleaching teeth that have been previously sealed.
- The intense smell of BlancOne® SEAL volatilizes during drying within 1 minute.
- BlancOne® SEAL contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour. After eye contact rinse out with plenty of water and call an ophthalmologist. After contact with skin, wash with plenty of water and soap.
- Avoid contamination of the BlancOne® SEAL with dirty disposable brushes.
- Impressions with vinyl silicones can be made earliest 4 weeks after the treatment with BlancOne® SEAL.

Storage:

Do not store above 25 °C (77 °F)! Do not use after expiration date!

Store BlancOne® SEAL tightly closed in a well ventilated place.

**Keep away from children!
For dental use only!**

Warranty

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and SanaPro Dental GmbH's sole obligation shall be repair or replacement of the SanaPro Dental GmbH product.

Limitation of Liability

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Distributed exclusively by:
IDS Spa 17100 Savona - Italy
Via S. Cristoforo, 28/10
www.ids dental.it

www.blancone.eu/pro

Manufacturer:
SanaPro Dental GmbH
27570 Bremerhaven, Germany

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